Mölnlycke®

Granudacyn®

Wound gel

Manufacturer / Fabricant:  
P.G.F. Industry Solutions GmbH  
Katzmoosstr. 26a  
5161 Elixhausen  
AUSTRIA

CE0044

en	Medical Device	no	Medisinsk utstyr
de	Medizinprodukt	sl	Medicinski pripomoček
fr	Dispositif médical	bg	Медицинско изделие
es	Producto sanitario	ro	Dispozitiv medical
nl	Medisch hulpmiddel	sk	Zdravotnícka pomôcka
sv	Medicinteknisk produkt	tr	Tıbbi cihaz
it	Dispositivo medico	lt	Medicinos priemonė
fi	Lääkinnällinen laite	lv	Medicīniska ierīce
pt	Dispositivo médico	et	Meditisiiniseade
da	Medicinsk udstyr	ru	Медицинское изделие
el	ιατροτεχνολογικό προϊόν	hr	Medicinski proizvod
pl	Wyrób medyczny	sr	Medicinsko sredstvo
cs	Zdravotnický prostředek	is	Lækningatæk
hu	Orvostechnikai eszköz	ar	جهاز طبي

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en

## Product description

Granudacyn® Wound gel is a ready-to-use, clear gel for cleaning and moistening acute and chronic wounds as well as for keeping wounds moist. Cleaning of wounds prevents wound infection. In wounds that are already infected, adjuvant wound irrigation can help to significantly reduce the microbial load. This results from the mechanical effect of the irrigation, which together with moistening promotes wound healing.

Content: Water, sodium chloride, hypochlorous acid, sodium hypochlorite, colloidal silicate

## Indications for use

Granudacyn® Wound gel may be used for the thorough wound cleaning of:

- Acute or chronic wounds as well as surface wounds
- Cuts, abrasions and 1st and 2nd degree burns
- Critically colonised or infected wounds

## Precaution(s)

- Should not be combined with other wound cleaning products (such as enzymatic wound cleaners)
- Do not mix with other products or chemicals.
- Only use intact and undamaged packages.
- Oral ingestion, although harmless, is nevertheless not recommended.
- Do not apply onto or inject into tissue using pressure.
- Do not use Granudacyn® Wound gel in the event of hypersensitivity or suspected allergy to the components.

## Instructions for use

Granudacyn® Wound gel can be used by lay persons under supervision of health care professionals and is suitable for repeated use. Shake well before use.

Clean the wound and surrounding area with Granudacyn® Wound Irrigation solution and dab dry. Apply Granudacyn® Wound gel to the affected areas and cover with a suitable secondary dressing.

Granudacyn® Wound gel can be used with Mölnlycke Health Care foam dressings: Mepilex, Mepilex Border and Mepilex Border Flex.

## Special storage conditions and handling conditions

- Store the product between 3 and 25°C
- Protect from direct sunlight
- Once opened, use within 90 days
- Disposal should be handled according to local environmental procedures

**Other information**

Granudacyn® is registered trademark of Mölnlycke Health Care AB.

If any serious incident has occurred in relation to the use of Granudacyn® Wound gel, it should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Mölnlycke Health Care Pty. Ltd.